CACILIO20® RGTA®

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// Dernière révision: Mars 2018 (Ed. 02)

CACILIO20® RGTA®

Cutaneous chronic wound healing kit

Matrix therapy for regeneration

EN

INSTRUCTIONS FOR USE

(Composition)

Sodium salt of 1-6 poly(carboxymethylglucose sulphate) 0.1 mg/mL, sodium chloride 9 mg/mL, sodium hydroxide pH of 6.5 +/- 0.5, purified water. One kit for single use consists of one vial of CACILIO20® sterile solution and a blister pack containing one sterile gauze and two sterile forceps.

(General information)

CACILIO20® is an RGTA® (ReGeneraTing Agents) tissue matrix protective agent, which promotes natural skin regeneration. It is supplied in 3 presentations:
- 1x 1.5 mL vial of CACILIO20® and 1 healing set for a single application on a wound smaller than 35 cm².
- 3x 1.5 mL vials of CACILIO20® and 3 healing sets, for 3 applications on a wound smaller than 35 cm².
- 2x 5mL vials of CACILIO20® and 2 healing sets, for 2 applications on a wound between 35 cm² and 220 cm².

(Intended use)

CACILIO20® is a RGTA® (ReGeneraTing Agent), intended to improve the healing of chronic cutaneous ulcers by promoting the restoration of the extracellular matrix, with the consequent reduction in lesion induced pain.

(Indication)

It is indicated for chronic cutaneous ulcers caused by pressure, peripheral arterial disease (e.g. stage I/Verluche and Fontaine classification), or diabetes (including amputation), showing no tendency to heal after 6 months of standard care, or still unhealed after 12 months.

(Treatment)

CACILIO20® is intended to be applied twice a week (every 3 to 4 days), ideally when the dressing is changed, a more frequent use may reduce the therapeutic effect. Results may vary from patient to patient, your physician may adapt the duration of treatment accordingly. Results should be observable within 3 months of use.

(User instructions)

Normal hygiene precautions should be taken when treating a wound with CACILIO20® (e.g. handwashing, appropriate protective clothing, examination table sheet...). In case of a sharp debridement, follow current best practice procedures to ensure sterility and prevent cross contamination. Use on chronic skin wounds every 3 – 4 days (twice a week)

1

2

3

4

1. Prepare the wound area carefully to ensure the wound is free from exudate, fibrin clot and necrotic tissue. Rinse with saline. Do not use iodine, silver or other polycation-based solutions before using CACILIO20®.

In case of a sharp debridement, follow current best practice procedures to ensure sterility and prevent cross contamination.

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3. Carefully impregnate the wound with the soaked gauze.

4. After 5 min, remove the gauze and cover the wound with the usual dressing (such as paraffin gauze dressing and a non-adherent secondary dressing).

(Contraindications)

Do not use in patients with a history of allergy or anaphylactic reactions to any of the ingredients or heparinoids.

(Warnings)

Due to lack of clinical data, CACILIO20® is not indicated for use by pregnant or breastfeeding patients; nor patients under 18 years old.
Choking Hazard – Keep out of the reach and sight of children.
CACILIO20® vials and healing sets do not contain preservatives, and are single use. The product will not be sterile in case of re-use. To prevent contamination, do not re-use or attempt to re-sterilize. Do not pour the solution directly onto the wound. Do not inject. Do not swallow (if swallowed, rinse the mouth with water) or use in any way other than as prescribed (see instructions). Respect the dosage prescribed. If CACILIO20® is applied too frequently, it may reduce the therapeutic effect. Control any infections and clean the wound before treatment. Do not use on necrotic tissue. Do not use on highly purulent wounds. Do not use on patients with a history of hypersensitivity to any of the ingredients or to heparinoids. Individuals sensitive to heparinoids may have a local reaction, (redness, local swelling, pain, etc.), which should not affect the healing process. If symptoms persist, or worsen, discontinue use and seek medical advice.
CACILIO20® is not indicated for patients eligible for revascularization surgery. CACILIO20® is not indicated for patients using a non-removable discharge boot to treat diabetic foot ulcers.
According to the chemical reactivity studies and literature, CACILIO20® should not be used with:
- Locally applied aminoglycoside antibiotics (gentamicin, neomycin, etc.)
- Solutions containing silver salts, iodine, copper, gold or polycation based products such as quaternary ammonium disinfectants (e.g. benzalkonium chloride) as these ions can inhibit the molecule.

(Precautions)

Do not use if the packaging is damaged. Do not use any tools to open the packaging. In the event of unintended contact with the skin or contact with the eye, rinse with lukewarm running water.

(Users)

CACILIO20® should preferably be applied to a wound cleaned by sharp debridement by a health care professional. CACILIO20® is also effective when applied to a cleaned wound, free from exudate, fibrin clot and necrotic tissue, by the patient or their carers. CACILIO20® should be used under the supervision of a health care professional.

(Storage)

Store in a dry place between 2°C and 25°C. Brief and occasional exposure to higher temperatures (less than 40°C) does not harm the integrity of the product. Do not use after the expiry date indicated on the pack.

(Manufacturer of CACILIO20®)

Vial of CACILIO20® solution.

OTR3, 4 rue Française, 75001 Paris, France
contact@otr3.com
Tél: +33(0)1 83 64 23 00
Fax: +33(0)9 55 72 44 53

LABORATOIRE CAT

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45260 LORRIS, France

// Last revision: March 2018 (Ed. 02)

[illegible][illegible][illegible]